


MINI-BAG Plus System

Designed for use with only standard single dose powdered 20 mm drug vials (Use Aseptic Technique)


QUICK TIPS:

- Do not twist bag when docking vial (internal spike may become damaged)
- Once docked, vial remains attached to maintain the closed system
- Before administration, ensure drug is fully dissolved and vial is empty
- If fluid drips back into vial after reconstitution is complete, discard system


> ASSEMBLY DIRECTIONS

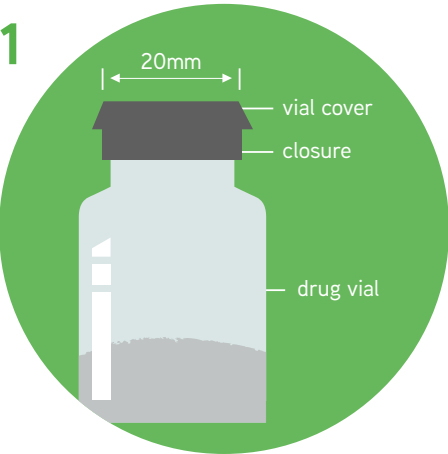

- Remove vial cover
- Disinfect stopper

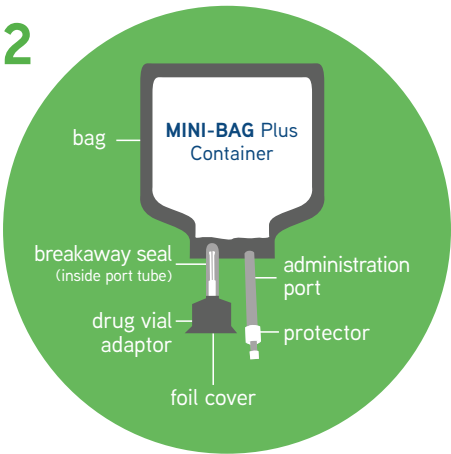

- Peel off foil cover
- Inspect adaptor for moisture
- Discard if moisture found

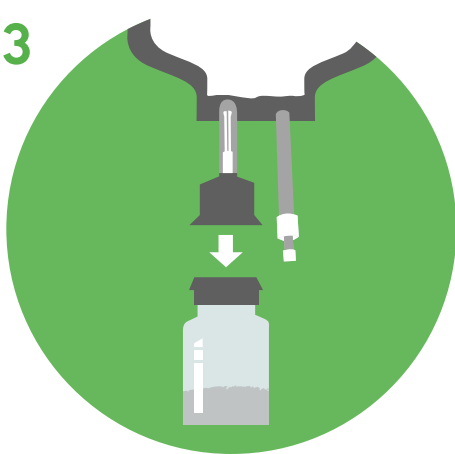

- Place vial upright and hold firmly
- Push adaptor down to snap vial in place
- **DO NOT TWIST**
- Pull vial to ensure fully seated

> RECONSTITUTION DIRECTIONS



- Squeeze bag and check vial
- Use only if vial fully seated and dry
- Bend up then down to break seal




- Hold bag with vial down
- Squeeze solution into vial until half full
- Shake to suspend drug in solution




- Hold bag with vial upside down
- Squeeze bag to force air into vial
- Release to drain suspended drug in vial
- Repeat steps 5 and 6 until vial is empty of drug and solution is thoroughly mixed
- Ensure drug is completely dissolved
- **DO NOT REMOVE DRUG VIAL**


> ADMINISTRATION DIRECTIONS

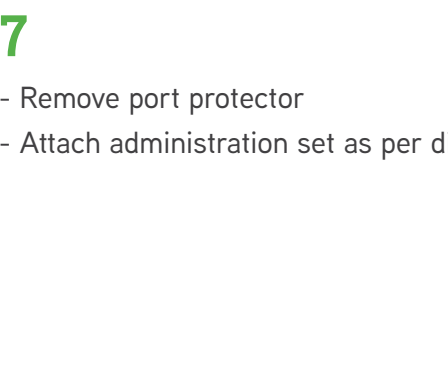

- Remove port protector
- Attach administration set as per directions

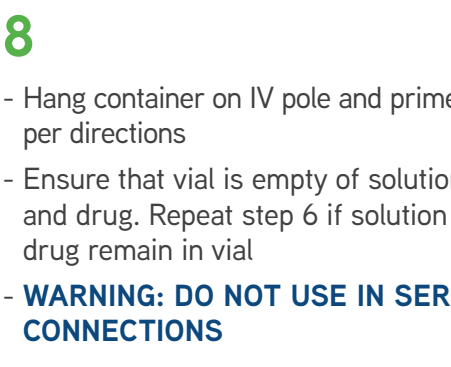

- Hang container on IV pole and prime set per directions
- Ensure that vial is empty of solution and drug. Repeat step 6 if solution and drug remain in vial
- **WARNING: DO NOT USE IN SERIES CONNECTIONS**

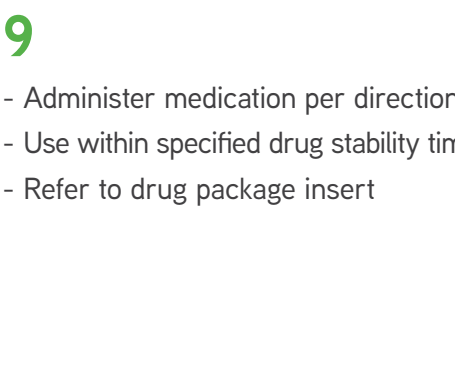

- Administer medication per directions
- Use within specified drug stability time
- Refer to drug package insert

To order the MINI-BAG Plus System, call 1-888-719-9955

Baxter Corporation, 7125 Mississauga Road, Mississauga, Ontario L5N 0C2