

Welch Allyn RetinaVue 700 Imager

Startup Guide

To get started, go to <https://www.hillrom.com/en/products/retinavue-700/> for information about using and servicing your product >>

- How To Videos
- Instructions For Use
- Service and Support
- Warranty Registration



1. RetinaVue 700 Imager	5. Lens cloth
2. USB cable	6. Wrist strap
3. Wall adapter plug and AC power cord	7. Instructions for use
4. Lithium-ion battery	













Baxter, RetinaVue and Welch Allyn are trademarks of Baxter International Inc. or its subsidiaries.

Baxter Technical Support
[baxter.com/contact-us](https://www.baxter.com/contact-us)


Mat. No. 778404, 80031283 Ver. A
Revision date: 2025-01







(91)778404

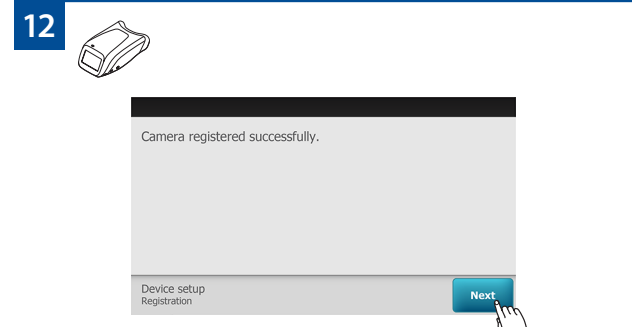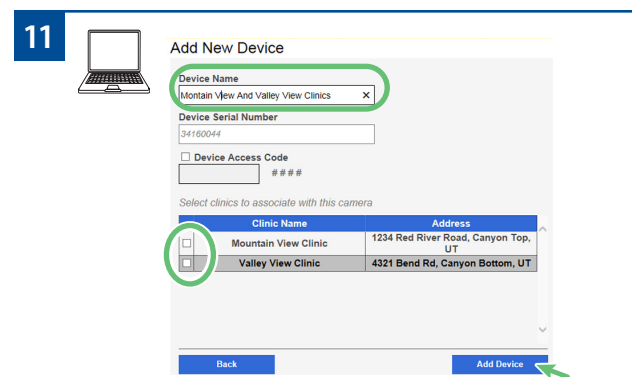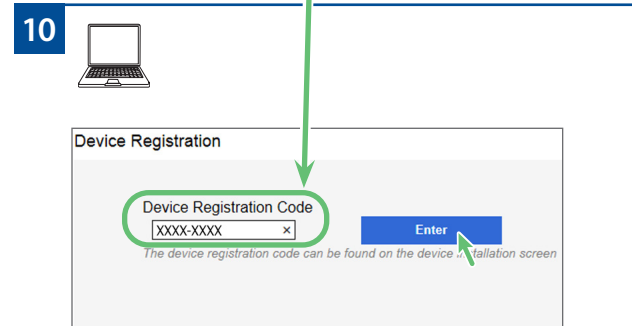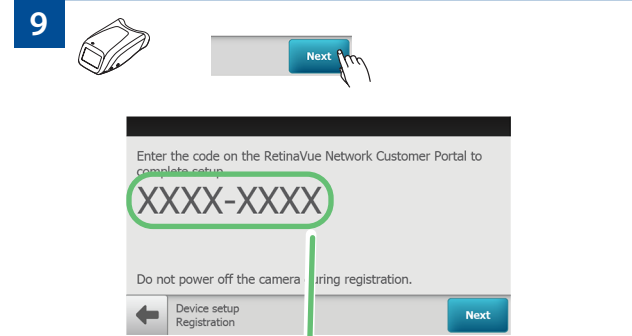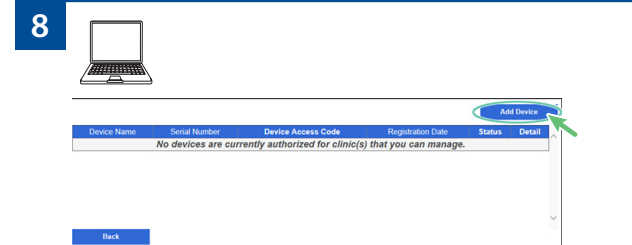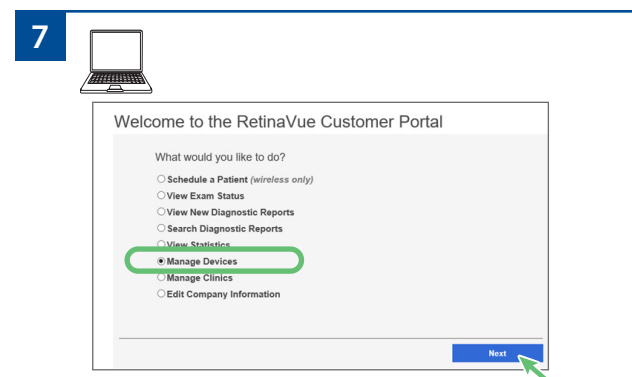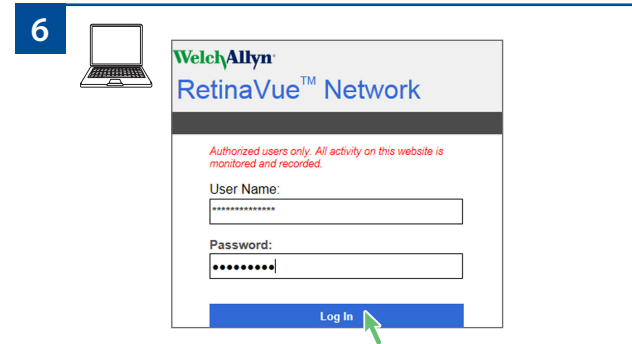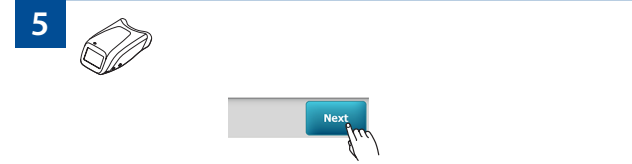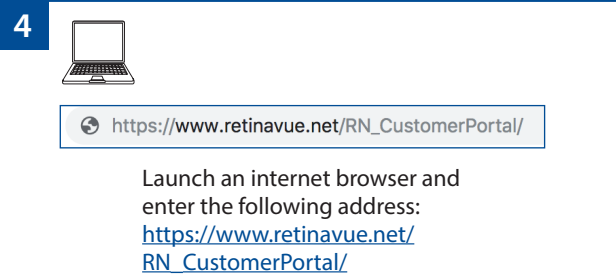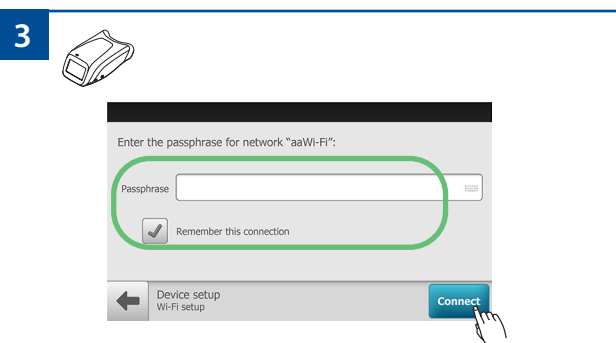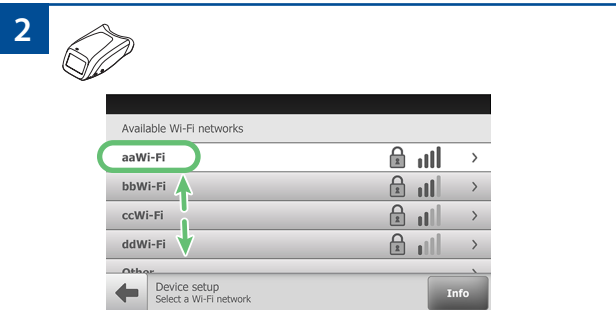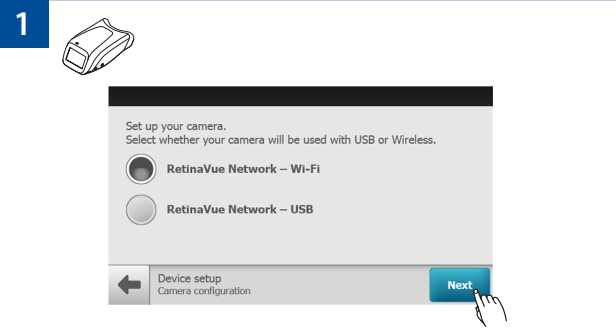

Battery assembly and charging

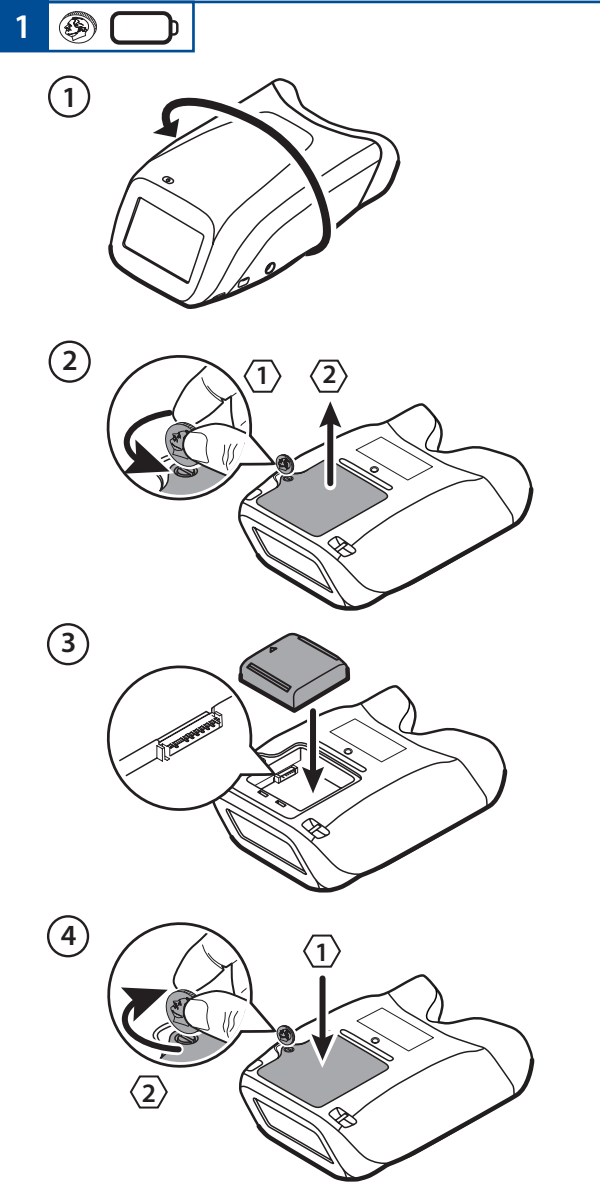

Startup imager

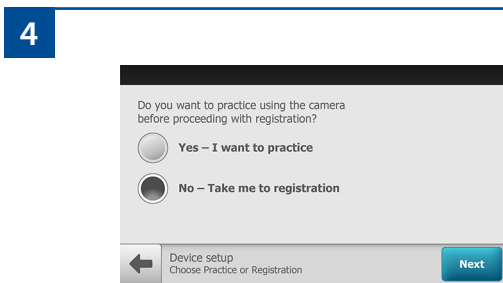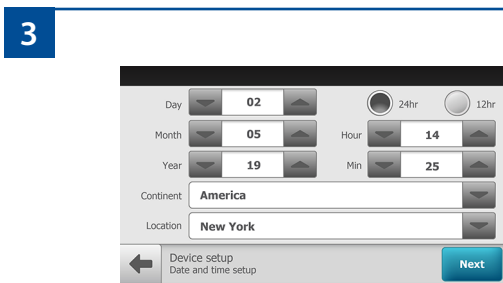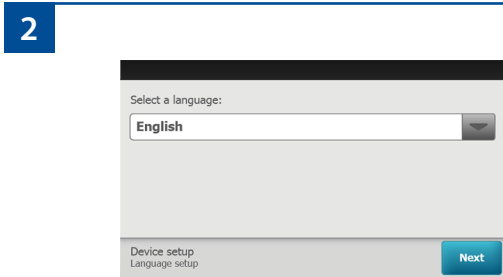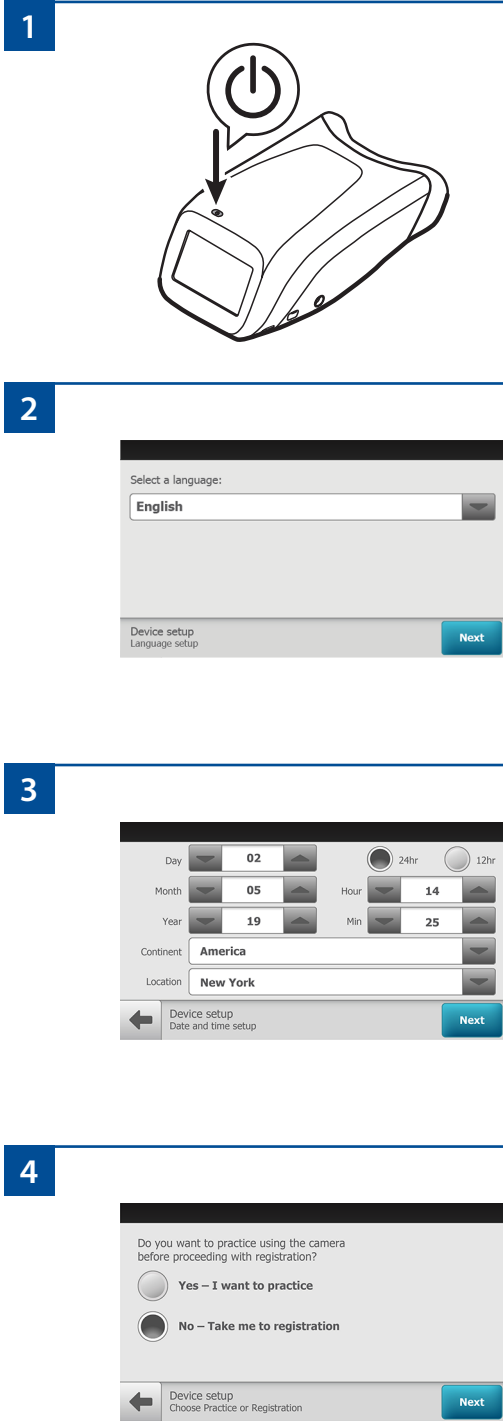

RetinaVue Network registration for USB or Wi-Fi connectivity



Call: 1.865.622.8380 to set up an initial account with the **RetinaVue** Network.



You will receive an email via Adobe's eSign service that includes your **RetinaVue** Professional Service Agreement.



Electronically sign and submit your **RetinaVue** Professional Services Agreement.



You will receive an email from **RetinaVue** Network System with the initial username and password and a link to the **RetinaVue** Customer Portal.



Go to the **RetinaVue** Customer Portal and complete your **RetinaVue** Network account registration.

For Wi-Fi connection, go to the next page.

Additional registration steps for USB connectivity



DOWNLOAD

Upon completion of your account registration, download the **RetinaVue** Software from the **RetinaVue** Customer Portal onto the PC you will be using to transfer images to the **RetinaVue** Network.



Launch the **RetinaVue** Network software. Enter your **RetinaVue** Network Software Activation key into the **RetinaVue** Network software.

USB connection

