

Connect to NEO-x-1
and NEO-x-2 Cuffs
x = Size 1, 2, 3, 4, or 5



Connecting Welch Allyn devices to new ISO compliant Neonatal NIBP cuffs.

Device	Neonatal NIBP Capability?	Neonatal NIBP Monitor Tube	Neonatal Monitor Tube ID	Connector Hose Barb ID Required	Connector Part Number
	No	N/A	N/A	N/A	N/A
	No	N/A	N/A	N/A	N/A
	Yes	008-0265-01	0.085" (2.2mm)	3/32" (2.4mm)	5082-302*
	No	N/A	N/A	N/A	N/A
	No	N/A	N/A	N/A	N/A
	Yes	7000-33	Neonatal NIBP tube comes with ISO complaint Neo connector		
		6000-33	0.085" (2.2mm)	3/32" (2.4mm)	5082-302*
	Yes	7000-33	Neonatal NIBP tube comes with ISO complaint Neo connector		
	Yes	008-0265-01	0.085" (2.2mm)	3/32" (2.4mm)	5082-302*
		5082-244 5082-245 5082-246	0.125" (3.2mm)	1/8" (3.2mm)	5082-301*
		008-0265-01	0.085" (2.2mm)	3/32" (2.4mm)	5082-302*

**Cut ¾" (19mm) off cuff end of tubing to remove luer-slip female connector. Insert hose barb of connector until fully seated. Warm tube end or dip hose barb in 70% isopropyl alcohol to ease assembly process.*