

Pocket LED & Pocket Plus LED

Only version
available in US/C

	Pocket LED	Pocket Plus LED
Shared Features		
Sure Color LED	✓	✓
Rugged (superior drop test survival rate to competition)	✓	✓
Lightweight	✓	✓
Oto Head Differences		
Light output	3 Lumens	6 Lumens
Oph Head Differences		
Neutral density light filter switch		✓
Brow Rest	Acrylic	Soft Rubber
Set Differences		
Otoscope window bumpers	Available for extra charge	✓
Handle bottom bumper		✓
Warranty	3 yrs	5 yrs
Case	Soft	Hard

Product ID

Customizable, Rugged, Stylish



Vanilla



Blueberry



Plum



Onyx

Cases

Soft Case

(included with Pocket LED set)



Hard Case

(included with Pocket Plus LED set)



Accessories

First scope to feature window and handle bumpers for added durability and style




Accessory Kit*




***Accessory Kit included in
Pocket Plus LED Set only**

Pocket LED Ophthalmoscope Controls

Doctor Side

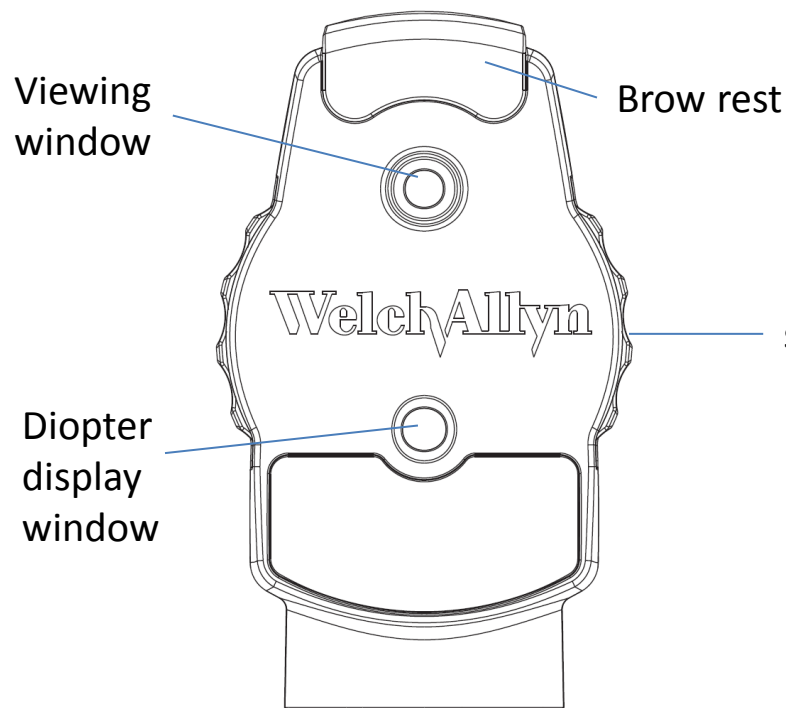

Patient Side

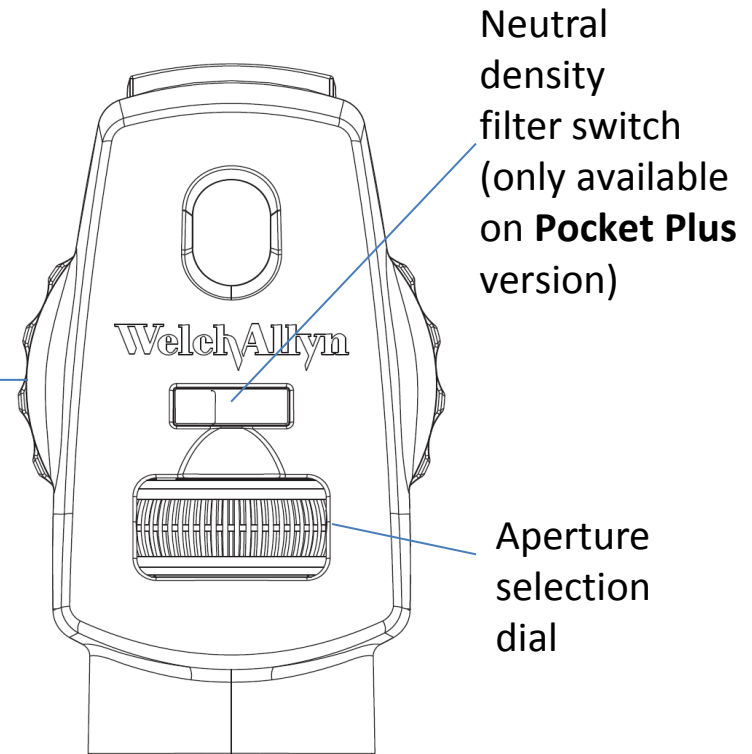

Part Numbers

Nma i cr JCB lmr qmjb g l SQ-A

Nma i cr Njsq JCB m l jw qmjb g l SQ-A

Pocket LED Product Line Customer-Facing PN's and Descriptions			Case		Bumpers		Ear Tips	
			Soft Case	Hard Case	Complete Accessory Kit (1)	2 Color Accessory Kit (2)	Tube of Reusable and Disposable Tips (3)	Bag of 4 Reusable Ear Tips (4)
SETS	+	92871-BLK	Pocket LED Set/Onyx w/ Soft Case	✓			✓	✓
		92871-WHT	Pocket LED Set/Vanilla w/ Soft Case	✓			✓	✓
		92871-PUR	Pocket LED Set/Plum w/ Soft Case	✓			✓	✓
		92871-BLU	Pocket LED Set/Blueberry w/ Soft Case	✓			✓	✓
		92880-BLK	Pocket Plus LED Set/Onyx w/ Hrd Case		✓		✓	
		92880-WHT	Pocket Plus LED Set/Vanilla w/ Hrd Case		✓		✓	
		92880-PUR	Pocket Plus LED Set/Plum w/ Hrd Case		✓		✓	
		92880-BLU	Pocket Plus LED Set/Blueberry w/Hrd Case		✓		✓	
OTO ONLY	+	22870-BLK	Pocket LED Otoscope/Onyx w/ Handle					✓
		22870-WHT	Pocket LED Otoscope/Vanilla w/ Handle					✓
		22870-PUR	Pocket LED Otoscope/Plum w/ Handle					✓
		22870-BLU	Pocket LED Otoscope/Blueberry w/ Handle					✓
		22880-BLK	Pocket Plus LED Oto/Onyx w/ Handle	✓		✓	✓	
		22880-WHT	Pocket Plus LED Oto/Vanilla w/ Handle	✓		✓	✓	
		22880-PUR	Pocket Plus LED Oto/Plum w/ Handle	✓		✓	✓	
		22880-BLU	Pocket Plus LED Oto/Blueberry w/ Handle	✓		✓	✓	
OP ONLY	+	12870-BLK	Pocket LED Oph/Onyx w/ Handle					
		12870-WHT	Pocket LED Oph/Vanilla w/ Handle					
		12870-PUR	Pocket LED Oph/Plum w/ Handle					
		12870-BLU	Pocket LED Oph/Blueberry w/ Handle					
		12880-BLK	Pocket Plus LED Oph/Onyx w/ Handle	✓		✓		
		12880-WHT	Pocket Plus LED Oph/Vanilla w/ Handle	✓		✓		
		12880-PUR	Pocket Plus LED Oph/Plum w/ Handle	✓		✓		
		12880-BLU	Pocket Plus LED Oph/Blueberry w/ Handle	✓		✓		
Accessories		106081	Pocket Plus LED Accessory Kit, Complete					
		21103-U	Tube of assorted ear specula, 4 sizes					
		05928-U	Soft Zip Case for 2 Pocket LED Scopes					
		410521	Hard Case for Pocket LED Scopes and Acc	For internal reference only - - Not orderable by customers/ dealers				

Pocket LED FAQ's

SureColor LED Lamp

Q. How long does the LED lamp last?

A. *The LED lamp will generally provide over 300 hours of continuous on time*

Q. Is the LED lamp replaceable?

A. *The LED lamp is built into the device and is not replaceable.*

Q. Why is SureColor better?

A. *LED illumination can be so bright that it washes out structures in the eye and ear, or casts them in a blue or green hue. SureColor LED's closely match our halogen lamps in color, temperature, and intensity, so that things look the way physicians are used to seeing them.*

Pocket LED Otoscope (basic and Plus versions)

Q. Can the light intensity be controlled?

A. *No. The illumination is fixed at optimal intensity and can only be turned off or on*

Q. Can Pneumatic Otoscopy be performed with the Pocket LED Otoscope?

A. *Yes. The Welch Allyn #21504 Insufflator will connect to the port on the side of the otoscope*

Pocket LED FAQ's (cont.)

Pocket LED Ophthalmoscope

Q. How many filters are included?

A. *The Pocket LED Ophthalmoscope has 3 filters: red-free, cobalt blue, and neutral density*

Q. What is a neutral density filter?

A. *This filter can be used in situations where the user wants to reduce any glare coming back off the eye. The switch is located on the front (patient side) of the scope.*

Q. Which different apertures are available?

A. *7, as follows: Micro-Spot, Small Spot, Large Spot, Slit Aperture, and Fixation Aperture, Cobalt Blue, and Red-Free*

Q. What is the lens diopter range, and how many total lenses are there?

A. *Range is -20 to +20, and there are 19 lenses, as follows:*

-20, -15, -10, -8, -6, -4, -3, -2, -1, 0, 1, 2, 3, 4, 6, 8, 10, 15, 20

Pocket LED FAQ's (cont.)

Service and Warranty

Q. Are any parts replaceable?

A. *Only the handle bumpers, window bumpers, and cases*

Q. What are the warranties:

A. *Pocket LED Otoscope and Ophthalmoscope have a 3 year warranty*

Pocket LED Plus Otoscope and Ophthalmoscope have a 5 year warranty

General

Q. Are any of the parts inter-changeable with other Welch Allyn pocket scopes?

A. *No. The new heads will not work on any other handles*

Q. Where can I find the lot code on the product?

A. *Every handle and head has a sticker including a GTIN#*

Q. Are there any trade-in programs for my current pocket set?

A. *No*